Medisch hulpmiddel CE 0481

Wrat weg

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Medical Device CE 0481

Wart away Painless* wart treatment

- For the treatment of common warts on hands and feet
- Dab only once
- For adults and children (4 years and over)

* Note: When applied correctly by an adult.

General advice: Carefully read the instructions for use prior to using Wart Away, even if you used this medical device before. The information may have been amended according to new research and findings in the time since you opened your last package. Take care when using Wart Away in order to obtain a good result. Keep these instructions for use in case you need to consult them again.

Application: Wart Away is a medical device for the treatment of common warts (verruca vulgaris). It contains monochloroacetic acid, a substance having a degrading action on the skin. The wart will dissolve, layer after layer. Monochloroacetic acid will make sure that the cells containing the wart virus disappear.

The benefits: One benefit of using Wart Away is that a single dab on the wart does not hurt, which means that it can also be used for children (when applied correctly by an adult).

- Suitable for children from the age of 4;
- For the treatment of common warts on hands and feet;
- Only 1 treatment per week;
- Painless treatment*
- Easy and simple to use.

WART AWAY ONLY TO BE USED WITH:

Common wart (verruca vulgaris)

- Primarily occur on hands and feet;
- Same colour as the skin;
- Circumference varies from less than a millimetre to more than a centimetre;
- Several warts usually occur simultaneously;
- Contamination often occurs through direct contact, but also through towels or door handles;
- Scratching and nail biting increase the risk of contamination.

DO NOT USE WART AWAY WITH:

Senile wart (verruca seborrhoica)

- Can come in different sizes and colours;
- Average circumference is 0.5 to 1 cm;
- Often start as a skin-coloured, yellow-brown, greyish-brown patch. Later on, become bigger and fatter;
- Appear as if they are lying on the surface of the skin and are easy to scratch off, but are actually firmly attached to the skin;
- They can occur anywhere on the body, except for the palms of the hands and soles of the feet, and the lips. Most likely to occur on the chest, the back, the face, and the neck.

Plantar wart (verruca plantaris)

- Found on the bottom/sole of your foot;
- Usually flat and surrounded by rough, thickened skin;
- Resemble calluses;
- Can contain black dots or tiny seeds;

Instructions for use: Prevent burns:

DO'S:

- Application by adults;
- Use only on hands and feet! Do not use on other body parts;
- Rub Vaseline or zinc ointment into the surrounding skin to protect the skin;
- Place the bottle on a solid underground that is able to resist the caustic fluid;
- Dab the wart/verruca just once when applying;
- Dab the wart just once per week;
- Allow the liquid to come into the contact with the wart/verruca only;
- Avoid contact with the skin;
- Allow the liquid to dry properly after applying;
- Leave the wart/verruca uncovered;
- The result is visible after 1 week;
- Keep out of the reach of children.

DON'TS:

- Apply by individuals under the age of 18;
- Apply to warts/verruca on any part of the body other than the hands or feet;
- Apply to any wart that is wholly or partially under a nail;
- Do not use on warts that occur completely or partially on the sole of the feet (plantar warts);
- Allow the liquid to come into contact with healthy skin;
- Apply when your hands are shaking or your fingers are stiff;
- Dab on more than once when applying;
- Cover the wart/verruca after applying the liquid;
- Apply on birthmarks, water warts, open wounds, hyperpigmentation, senile warts, plantar warts, dark-coloured hairy or weird looking warts;
- Apply during pregnancy or lactation.
- Repeat treatment without a one-week interval.

How to use (Application only by adults): Please apply Wart Away with utmost care. The solution is caustic. Be extra careful when applying it to parts on your hands or feet that are more difficult to reach. This particularly applies to persons with a limited mobility or in case of disturbed fine motor functions in the system (trembling hands, stiff fingers).

1. Wart Away has a child proof cap that can be opened as follows: Place the vial on a solid underground that is able to resist the caustic fluid. Avoid contact with clothes, textile or table-cloths. You can open the vial by pressing down the cap and turning it anticlockwise simultaneously.

2. Before removing the cap entirely from the vial, apply Vaseline or zinc ointment generously to the healthy skin surrounding the wart.
3. Remove the cap from the vial.
4. Then, apply a small amount of Wart Away to the wart using the spatula (do not allow the fluid to drip). Make sure that you only apply Wart Away to the wart and do not dab the wart for more than one time.
5. Allow it to dry for a moment and do not cover the wart. Carefully close the bottle after use. Wart Away should not be applied more than once per week.

The fluid will need a week to dissolve the top layer of the wart, so you will not see an immediate result after application. Depending on the size and thickness of the wart, it may be necessary to repeat the treatment with Wart Away after one week. Do not repeat treatment within a one-week interval.

Composition: Monochloroacetic acid(50%), purified water(50%).

Content: 2 ml bottle

Do not use Wart Away:

- If you are oversensitive to one or more of the ingredients;
- In the face (the fluid should never make contact with the eyes or the mucous membrane in the nose or mouth);
- For genital warts or in the genital area;
- On birthmarks, water warts, open wounds, moles, naevus, (brown) warts, plantar warts, darkly coloured, hairy or strange looking warts;
- On red, irritated or infected warts;
- Immediately after a surgical removal of warts, freezing of warts (cryotherapy) or a laser treatment. This particularly applies when the skin is red or painful after the treatment. In this case you will have to wait for at least 4 weeks, until the redness or inflammation of the skin has fully disappeared. Only then you can start using Wart Away;
- During pregnancy or while breastfeeding;
- Do not use on warts which occur completely or partly underneath the nail.
- Do not use on warts that occur completely or partially on the sole of the feet (plantar warts).

Be extra careful when using Wart Away:

- In case of doubts concerning the diagnosis (the change in the skin can not be clearly explained as a wart);
- In case of immune deficiency (e.g. when you are using cytostatic medication);
- In case of diabetes, or other complaints such as vascular diseases, as a result of which wounds will heal less easily.

In these cases, Wart Away may not be used before a doctor has been consulted.

Warnings:

- Only intended for external use;
- Use with utmost care. The fluid of Wart Away is corrosive;
- If the healthy skin makes contact with Wart Away, please rinse it with water. If, in spite of these precautions, the skin becomes irritated, it will have to undergo the same treatment as burns;
- Contact your general practitioner, if your complaints continue to exist;
- Properly close the bottle after use and store it in an upright position;
- Keep out of reach of young children;
- Use by children, the elderly and less able must be supervised by an adult;
- Avoid contact with clothes, textile, table-cloths or skin;
- Stop using if blisters occur.
- Dab wart no more than once and repeat only if necessary after a one-week interval.

Use in combination with drugs: As far as we know, there are no interactions with drugs.

Possible side effects: Wart Away may cause some side effects, although not everyone will experience these side effects. The side effects that could occur when using Wart Away are: irritated skin, burning sensation, skin redness or oversensitivity reactions. If any of these side effects becomes serious, or if you notice any side effects not listed in these instructions for use, please consult your doctor or pharmacist.

15°C 25°C

Storage instructions: Store at a temperature of 15-25°C in a dry and dark place in the original container and out of reach of young children. Do not use after the expiry date stated on the packaging.

Medical device: Wart Away is a class IIa medical device in accordance with the Medical Device Directive 93/42/EEC.

Final date of use: We advise against using Wart Away after the expiry date, which is stated on the packaging after 'expiry date'. Wart Away can be kept for 1 month after opening.

Date on which these instructions for use were last revised:
October 2024 • 038599-V2.9

PK Benelux BV
Vluchttoord 17, 5406 XP Uden, NL
 NL: 0900 582 68 48 • B: 0800 731 83

www.lucovitaal.nl

MD **CE 0481**

Index

	Not for pregnant women or while breastfeeding		Keep out of reach of Children
	Not for children aged 3 years and younger		Warning: Corrosive substance
	Caution		Consult instructions for use
	CE Mark Notified Body number 0481		Medical device
	Batch code		Manufacturer
	Use-by date		Temperature storage limit