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US PATENT D 540,462

AM10121-3

GB THE AGENT AGAINST SNORING AND BREATHE MORE FREELY THROUGH YOUR NOSE

Using the Airmax Improves the air flow and nasal respiration with the following results:

- Reduces "stuffed nose" sensation (the common cold, allergies and pregnancy)
- Optimized air flow through the nose reduces snoring
- Optimized air flow through the nose lets you enjoy exercising more
- If you sleep better you wake up more rested!
- Reduced dependency on nose drops

WHY THE AIRMAX?

The Airmax is the latest breakthrough in the field of 'nasal dilators'.

The shape of the Airmax was designed using aerodynamic expertise. The function has been tried and tested by ENT doctors.

Placing the Airmax inside the nose results in the narrowest part of the nasal entrance being widened.

The Airmax is made of a soft, odorless and medically certified plastic (S.E.B.S.) which makes using the Airmax extremely comfortable. The profile of the Airmax works very effectively and increases the air intake.

The function of the nose; warming up, cleaning and moisturizing the air inhaled is maintained or improved.

INSTRUCTIONS FOR USE

The Airmax must be rinsed with water both before and after use. When using the Airmax for the first time, we recommend the use of a mirror to help you position the device symmetrically. The Airmax has been properly positioned when the legs widen the nasal entrance and when the 'connecting bridge' is touching your nasal septum. Please refer to the illustration.

If you use the Airmax Sport, see to it that the small 'barbs' stay behind the nose pillar of the nasal septum. This way the Airmax doesn't drop out when you are working out.

First test both sizes to see which one has the most positive effect on your breathing. Start with size S and proceed as described earlier. Then try size M: If size M and size S both fit you are able to use both sizes. If size M leaves too much space between your nasal septum and the 'connecting bridge', size M is too big for you and you should use size S.

The Airmax can be cleaned with water and soap.

The Airmax can only be used by 1 person to prevent transfer of diseases like the common cold.

Use the Airmax at least 2 weeks in a row to get used to it. Your nose is not used to the shape of the Airmax yet and needs to settle.

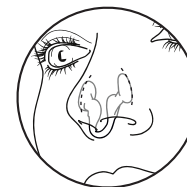
THE AIRMAX CAN BE USED FOR UP TO 3 MONTHS!

△ WHEN SHOULD YOU NOT USE THE AIRMAX?

When you have trouble placing the Airmax in your nose because your nose is not symmetrical or after you have placed the Airmax you still don't notice an improvement in breathing, we advise you to see a ENT doctor. If as a result of regularly using the Airmax your skin and mucous membrane become irritated, this could be an allergic reaction to the plastic. To date no allergic reactions are known.

△ ATTENTION!

The Airmax is only suitable for positioning inside the nose. The Airmax is not suitable for children under the age of 12. Keep the Airmax out of reach of younger children to prevent choking.

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RhinoLab GmbH.
AIRMAX B.V.